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Tandatangan : _____
Cop rasmi syarikat:

www.mymedic.com.my

URGENT: FIELD SAFETY NOTICE
High Level Disinfectant 5L With 0.15% Peracetic Acid

PLEASE DISTRIBUTE THIS INFORMATION WITHIN YOUR FACILITY TO ALL USER OF HIGH LEVEL DISINFECTANT 5L WITH 0.15% PERACETIC ACID

April 1st 2026

Dear Sir / Madam,

Re: High Level Disinfectant 5L With 0.15% Peracetic Acid with potential issues of excessive odour, bubble, corrosion and discoloration.

The purpose of this letter is to inform you of the potential issue as stated above that may occur during solution application. The affected lot no. are listed in the table below:

Product	MDA Registration No	Item Code	Lot. No
High Level Disinfectant 5L With 0.15% Peracetic Acid	GC7485024-157847	HLD90404	HLD404-2504 HLD404-2505 HLD404-2509

Reason For The Action:

Mymedic Innovation Sdn Bhd has received complaints from Hospital Sultanah Bahiyah regarding the potential issue of High Level Disinfectant 5L With 0.15% Peracetic Acid identified above. Its may pose a risk that affects to user safety and instruments integrity.

- The odour became excessive leading to irritation and discomfort to users due to the confined workspace, particularly during activation and within the first 5 day.
- The solution contains visible bubbles. No potential risk was identified.
- Instruments made of incompatible materials and oxide coated instrument reacted with the solution, causing corrosion and discoloration.

Actions Required:

Users are advised to implement the following action to address the potential issue:

1. Excessive odour

- **Wearing personal protective equipment (PPE)**
Users are required to wear a suitable mask, face shield, glove & apron when handling the solution to minimise exposure to odour during activation and application.

- **Cover the soaking tray**
Keep the soaking tray covered when not in use.
 - **Workspace and ventilation**
Users are advised to handle the solution in a workspace with sufficient ventilation to ensure proper airflow and minimise exposure to odour, particularly in confined areas.
2. **Visible bubble**
Users can handle the solution as usual, following the instruction for use (IFU). No potential risk has been identified because that visible bubbles are a normal occurrence, harmless and do not affect the performance of the solution.
3. **Corrosion and discoloration**
Do not immerse instruments made of incompatible materials comprising copper brass and galvanized steel such as instrument Ear Speculum and Tumarkin Ear Speculum Black (coated with black oxide).

Measures Taken To Resolve The Issue:

- **Product use guidance**
Users are advised to refer Safety Data Sheet (SDS), Instruction For Use (IFU) and the safety information provided in Field Safety Notice (FSN).
- **Product review**
Mymedic Innovation Sdn Bhd has improved the formulation by adding alcohol and anti corrosion agent to mitigate the potential risk as above. Cut-off batch no is HLD404-2510.

We appreciate your understanding and cooperation with this Field Safety Notice and ask you to immediately instruct your personnel accordingly. Please ensure that this safety notice is placed in the System's instructions for use. Your personnel should maintain awareness over an appropriate defined period.

MYMEDIC INNOVATION SDN BHD 200601037776 (757536-K)

No. 8, Lorong Sukepi 1, Off Jalan Sukepi, Kawasan Perindustrian Jenjarom,
42600 Jenjarom, Selangor Darul Ehsan

+60 3 3191 4868 enquiry@mymedic.com.my

If you have sold this medical device and it is no longer in your possession, we kindly ask that you forward this safety notice to the new owner of this medical device. Please inform us about the new owner of the medical device.

The **Medical Device Authority** will be informed of this notice.

Sincerely Yours

Mohamad Farkhan Bin Damri
Director
Mymedic Innovation Sdn Bhd

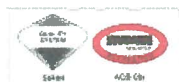
Contact person of this notification:

Department : Product Compliance

Telephone : 03-3191 4868 (ext 106)

Fax : 03-3181 7868

E-mail : productcompliance@mymedic.com.my

www.mymedic.com.my

MYMEDIC INNOVATION SDN BHD 200601037776 (757536-K)

No. 8, Lorong Sukepi 1, Off Jalan Sukepi, Kawasan Perindustrian Jenjarom,
42600 Jenjarom, Selangor Darul Ehsan

+60 3 3191 4868 enquiry@mymedic.com.my

Acknowledgement of receipt

I hereby confirm as the owner / responsible operator of the *High Level Disinfectant 5L With 0.15% Peracetic Acid* with the lot number HLD404-2504, HLD404-2505 and HLD404-2509 that I received the following document:

Field Safety Notice

High Level Disinfectant 5L With 0.15% Peracetic Acid with a potential issue of excessive odour, bubble, corrosion and discoloration.

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Signature : _____

Company stamp:

www.mymedic.com.my